

CEREC Connect

Download and Installation Instructions



Step 1: Log on to www.sirona-connect.net




Step 2: Select your language.




Step 3: Click on **Free CEREC Connect SW 4.X Download**




Step 4: Click on **CEREC Connect SW 4.2.3: exe file | 394 MB**

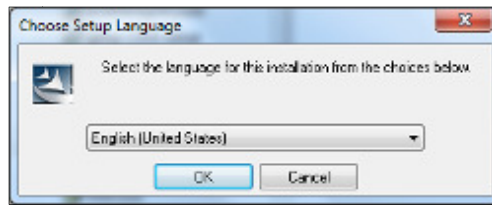


Step 5: Select the language for for installation in the Choose Setup Language dialog box, and click **OK**.

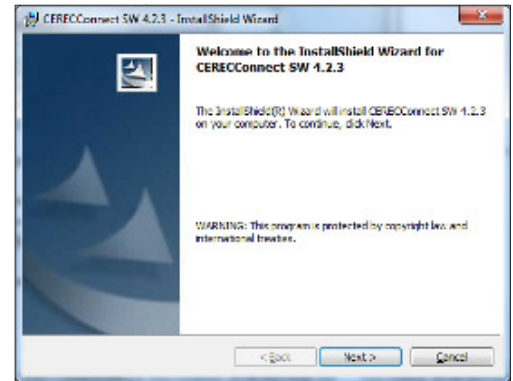


Step 6: On the Welcome to the InstallShield Wizard for CERECConnect SW 4.2.3 dialog box, click **Next**.

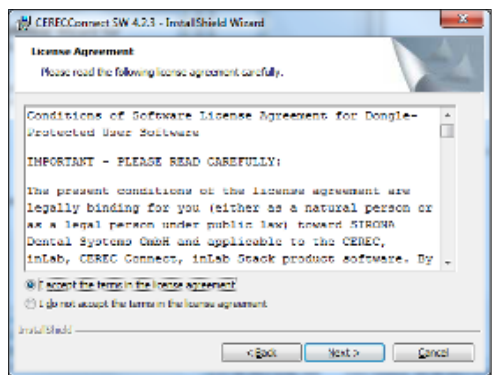


Step 7: On the License Agreement dialog box, choose **I accept the terms in the license agreement**, and click **Next**.

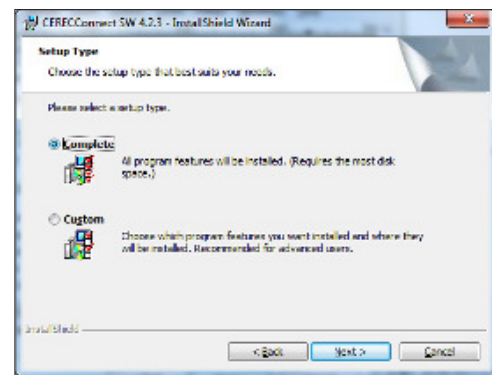


Step 8: On the Setup Type dialog box, select **Complete**, and click **Next**.

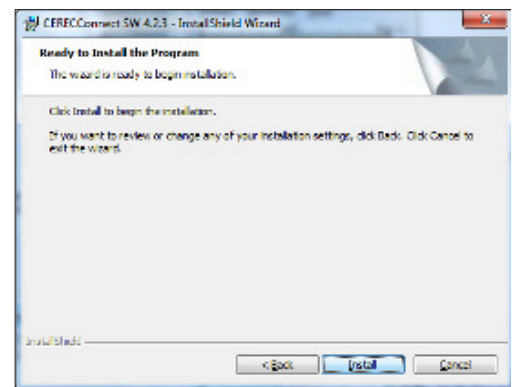

Step 9: Click **Install** to begin the installation.

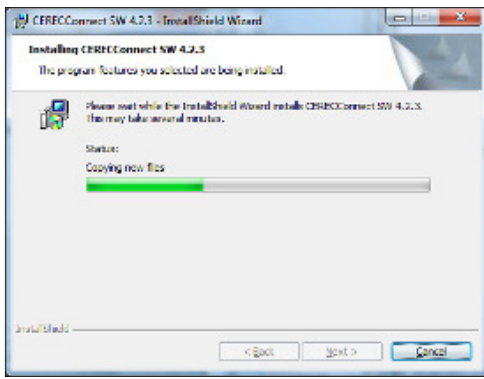

Step 10: The installation process may take several minutes to completely install.

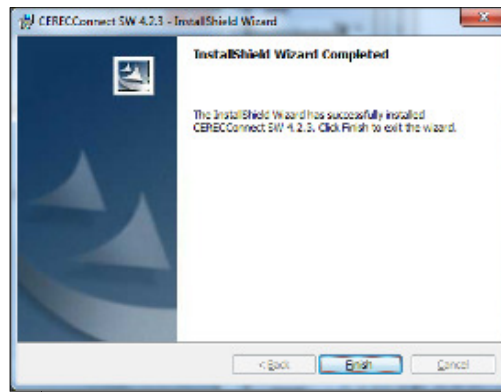

Step 11: When the InstallShield Wizard is complete, click **Finish**.